SPRAY NASAL

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NASAL SPRAY

Gently soothes and moistens dry and irritated noses.

What is newmed Nasal Spray and how do you use the product?

Dry, irritated upper airways with a tendency to encrustation frequently result in impaired breathing in both adults and children. The nasal mucous membranes can dry out, particularly in air-conditioned or heated rooms. However, encrustation of the nasal mucous membrane can lead to considerable impairment in children too.

The mild, isotonic, preservative-free solution used in newmed Nasal Spray can help here. newmed Nasal Spray contains natural spring water from Switzerland, supplemented by plant extracts of sage, peppermint and edelweiss, to combat dryness of the nose, as well as the encrustation of the nasal mucus membrane that is frequently associated with it, in a gentle and natural way.

newmed Nasal Spray is used:

- for gentle, supportive treatment of colds and blocked nose
- to moisten the mucus membrane of the nose in air-conditioned or heated rooms
- for cleansing in the case of dust exposure (fine dust etc.), to loosen encrustations

What do you need to note about using newmed Nasal Spray?

- It is important to observe the User Information!
- If necessary, newmed Nasal Spray can be used over a longer period of time. Seek advice from medical staff, such as your doctor or pharmacist, if symptoms have not improved after 14 days.

When must newmed Nasal Spray not be used?

- No unwanted effects are known when newmed Nasal Spray is used properly. newmed Nasal Spray should not be used in the case of known over-sensitivity to one or more of the ingredients.
- newmed Nasal Spray should not be used on infants and toddlers, as it is possible that the peppermint oil which it contains can trigger the Kratschmer-Holmgren reflex (respiratory arrest) when it is used in the area of the face (above all in the vicinity of the nose).
- If symptoms are persistent or recur frequently, seek advice from medical staff.

When should you take special care when using newmed Nasal Spray?

- No special precautions are necessary if used properly.
- If you are suffering symptoms following an ENT operation or following an injury to the nose, you should seek advice from a doctor.
- Let medical staff know if you are suffering from any illnesses, have any allergies, or are taking other medicines. This includes medicines that you have bought yourself.

How do you apply newmed Nasal Spray?

For adults and children of 2 years of age and over, spray 1-2 jets into each nostril several times a day, as required. For children under 2 years of age: spray 1 jet into each nostril, once a day.

Instructions for use:

1. Remove the protective cap and actuate the pump several times until a fine spray vapour appears.
2. Then insert the spray opening into one nostril and pump once or twice. Hold the bottle upright whilst doing so. Breathe in gently through the nose during spraying.
3. Repeat the procedure in the other nostril.

For reasons of hygiene and to avoid infections, it is recommended that you wipe the nose adapter with a dry cloth after use, and close it again with the protective cap. Furthermore, it should be used by only one person.

What side effects can newmed Nasal Spray have?

To date, no side effects have been observed when used properly.

What else must be noted?

- Store this product out of reach of children!
- Protect it from heat! Store at room temperature.
- The preparation may be used only up until the expiry date (EXP) stated on the packaging or the container. After opening, newmed Nasal Spray should be used within 6 months.

What does newmed Nasal Spray contain?

SODIUM CHLORIDE, LEONTOPODIUM ALPINUM (AQUA, GLYCERIN), GLYCERIN, SALVIA OFFICINALIS, MENTHA PIPERITA, AQUA

Manufacturer:

New Pharma AG, CH-8853 Lachen, Switzerland
Tel.: +41(0) 55 442 83 27, e-mail: info@newpharma.ch

CE Class I medical device in accordance with the Medical Devices Directive 93/42/EEC and 2007/47/EC.

User information version: December 2010

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