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SPRAY-ON PLASTER

For small, fresh, dry and clean skin injuries

What is newaid Spray-on Plaster and how do you use the product?

newaid Spray-on Plaster is the practical and convenient way to treat small skin injuries and abrasions. The water-resistant, transparent and breathable film protects the wound from dirt, water and germs, like a second skin. The film lasts for at least two days and then dissolves automatically.

newaid Spray-on Plaster:

- is used for small, fresh, dry and clean skin injuries
- protects, cools and soothes the wound
- is dermatologically tested

What do you need to note about using newaid Spray-on Plaster?

- It is important to observe the User Information!

When must newaid Spray-on Plaster not be used?

No unwanted effects are known where newaid Spray-on Plaster is used properly.

Do **not** use newaid Spray-on Plaster in the case of:

- a known over-sensitivity to one or more of the ingredients
 - injuries in the area of the eyes and mucous membranes
 - infected, deep, weeping wounds, or wounds covering a large area, as well as burns.
- In these cases, seek advice from medical staff.

When should you take special care when using newaid Spray-on Plaster?

- No special precautions are necessary if used properly.
- Let medical staff know if you are suffering from illnesses, have any allergies, or are taking other medicines. This includes medicines that you have bought yourself.
- Do not breathe in the spray vapour!

How do you apply newaid Spray-on Plaster?

Spray onto the skin area in question from a distance of 5-10 cm, as needed.

Instructions for use:

1. Shake before using for the first time.
2. Clean and dry the skin area in question.
3. Spray from a distance of 5-10 cm, directly onto small, fresh, dry and clean skin injuries. Repeat several times if necessary.
4. Afterwards, let the plaster dry for a minute.

What unwanted effects can newaid Spray-on Plaster have?

A slight burning sensation during use is possible.

What else must be noted?

- Store it away from children!
- The container is under pressure. Store at room temperature. Protect it from direct sunlight and temperatures above 50 °C.
- Do not open the contain with force, or burn it.
- Do not spray against flames, or onto glowing objects. Keep away from sources of ignition – do not smoke.
- Only completely emptied cans should be put into the domestic waste.
- The preparation may be used only up until the expiry date (EXP) stated on the packaging or the container. Do not use it after the expiry date!

What does newaid Spray-on Plaster contain?

ACETONE, ISOBUTANE, ALCOHOL DENAT., ACRYLATES COPOLYMER, AQUA, TRICLOSAN, LAURETH-9, BUTANE, PROPANE

Manufacturer:

New Pharma AG, CH-8853 Lachen, Switzerland
Tel.: +41 (0)55 442 83 27, e-mail: info@newpharma.ch

CE Class I medical device in accordance with the Medical Devices Directive 93/42/EEC and 2007/47/EC.

User information version: December 2010

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